

# Espiner Tissue Retrieval System

## Instructions for Use: Standard E-Sac (INS)

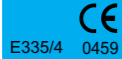### Illustration A: Standard E-Sac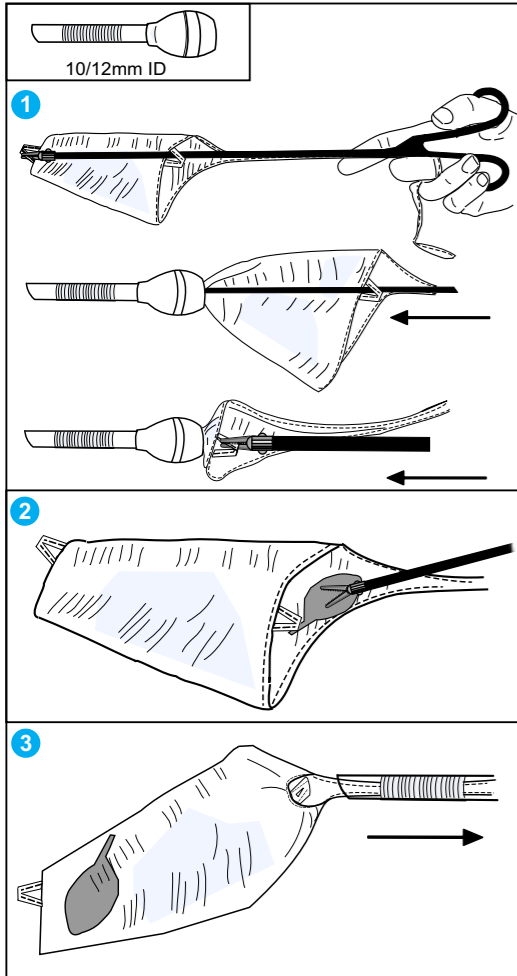

### ENGLISH (GB)

#### DEVICE DESCRIPTION:

**Device Name:** The Espiner Tissue Retrieval System

The Espiner Tissue Retrieval System is designed to facilitate the safe and secure removal of tissues or organs in the process of laparoscopic surgery. They are supplied sterile (ISO 11135) with a 5 year shelf life.

#### INDICATIONS FOR USE:

Retrieval System consists of a family of impervious sacs, which are sterile single use devices, that can be used alone, or with a dedicated introducer system, for the capture and removal of an organ, tissue or fluid from the body cavity during laparoscopic surgery.

#### CONTRAINDICATIONS:

There are no Contraindications for this device.

#### WARNINGS & PRECAUTIONS:

The device is only to be used by surgeon who are experienced with the specific techniques required by these medical devices.

Caution: Federal (USA) and European Union law restricts the device to sale, distribution and use by or on the order of a physician.

Use the drawstring by gripping the monofilament/suture approximately 15mm from the heatshrink.

The drawstring is designed for sac closure and retrieval of the sac mouth from the port site. Once outside the abdomen, continue to pull the mouth of the sac.

Caution: Do not cut, puncture or scrape the Espiner Tissue Retrieval System. Use tabs only for deployment of the device. Do not attempt to deploy by grasping the fabric, or by placing instruments inside the sac.

#### INSTRUCTIONS FOR USE -

##### \*Refer to Illustration A

- Insertion:** Place forceps through the tab at the sac mouth and grip the tab at the base of the sac. Push the tab through the trocar as far as possible. Use the tab at the sac mouth to push the rest of the sac into the abdomen.
- Tissue Capture:** Place the tissue inside the sac mouth. Use forceps to ease the tissue into the sac, ensuring the tissue is fully contained. *Note: Black stitching indicates inside of sac mouth for tissue capture*
- Retrieval:** Once the tissue is fully contained, pull the tail of the sac to bring the tissue and sac up through the incision. The sac mouth will close automatically against the trocar.

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